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GB Information for use product group NOBAKAWA®

Product description:

Gel swab for cryotherapy and thermal therapy

Composition of the gel (mass fraction)

Water, glycerol (20-45 %), carbomer, preservatives chloromethylisothiazolinone and methylisothiazolinone.

Usage as cold swab:

Cool the swab at max. -18 °C for about 2 hours. Do not use swab in the case of disturbed circulation or the Raynaud-syndrome.

Usage as warm swab:

Put the swab in max. 80 °C hot water for about 8 minutes. The swab can be heated in the microwave oven at max. 650 watt in intervals of max. 10 seconds. Please knead the swab thoroughly after each interval and smooth it out to achieve a proportionate heating of the gel. If necessary, repeat the interval until the requested temperature for application is achieved. Do not effect more than 3 intervals.

Instructions for reutilisation

The product is re-useable. To this end, remove NOBAKAWA® protective cover or cloth, then clean swab with customary aldehyde-free and amine-free surface disinfectant for plastic surfaces, and allow to air dry. The product may only be re-used if the foil is undamaged and without holes. The product must therefore be checked before each reutilisation. Use a new NOBAKAWA® protective cover or a washed cloth for each new usage.

Please always note:

- Prior to each use, put the swab into a NOBAKAWA® protective cover or wrap into a thin cloth to avoid cold or heat burns.
- For external use only.
- Do not use the swab if damaged.
- In the case of skin or eye contact, rinse the gel off with a lot of running water. If the gel has been swallowed, rinse the mouth thoroughly with water and drink plenty of water afterwards. If necessary, consult a doctor.
- Product is unfit for consumption.
- This product may be used by children from the age of 8 years or by persons with restricted physical, sensory, or mental abilities, or who have inadequate experience and / or knowledge, if they are supervised or have been instructed regarding its safe use and have comprehended the resulting risks.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Waste disposal:

Used material is disposed of in accordance with the requirements of statutory regulations. This is the responsibility of the respective institution.

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NOBAMED Paul Danz AG · Höltkenstr. 1-5 · D-58300 Wetter (Ruhr)