



APPEARANCE

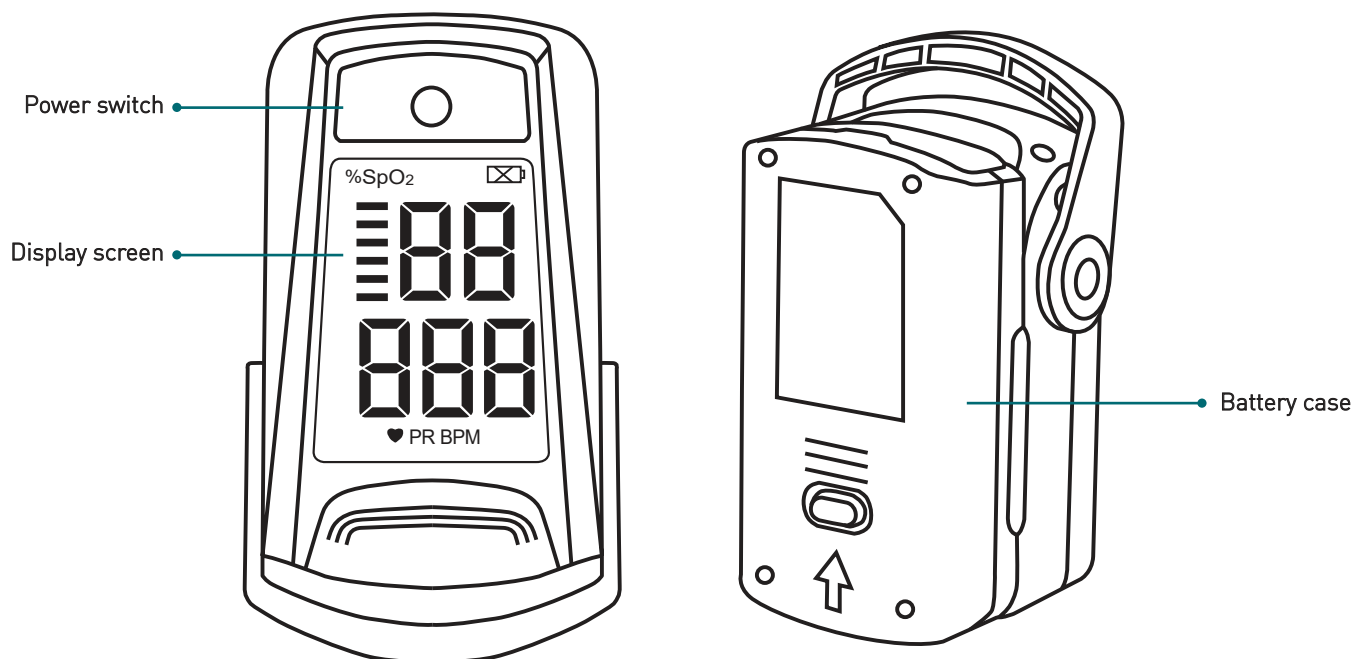

DISPLAY SCREEN

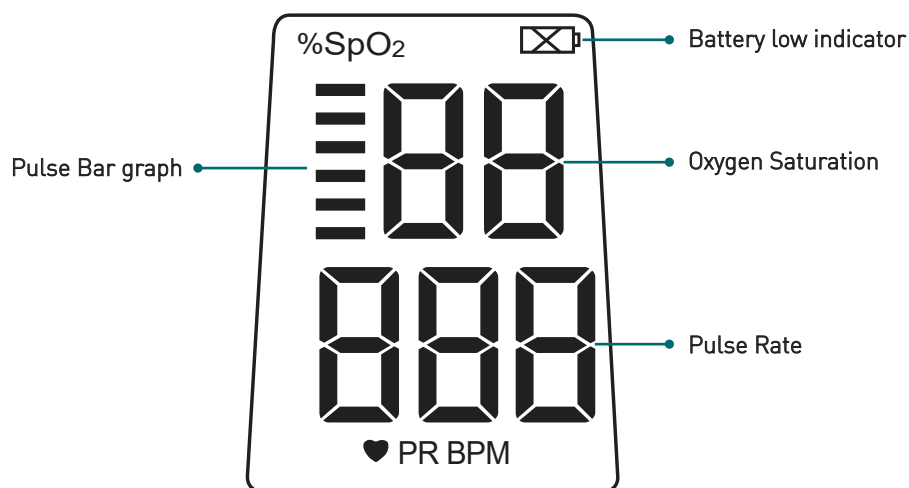


ORDERING INFORMATION


1905

ri-fox N Pulse Oximeter


USING THE DEVICE

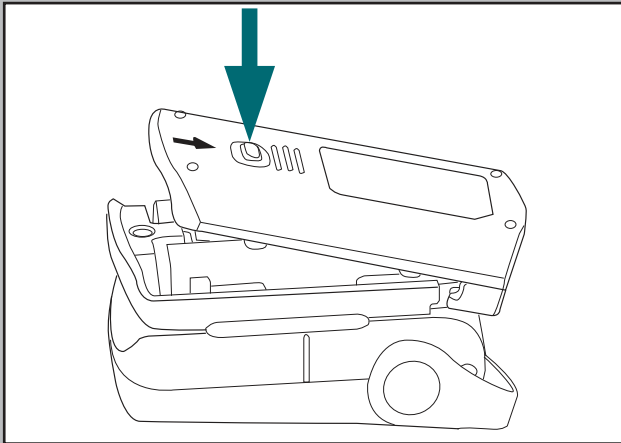


STEP 1. Install two AAA batteries into the battery cassette, and press the button down along the arrow and push the battery cover. Then close its cover.

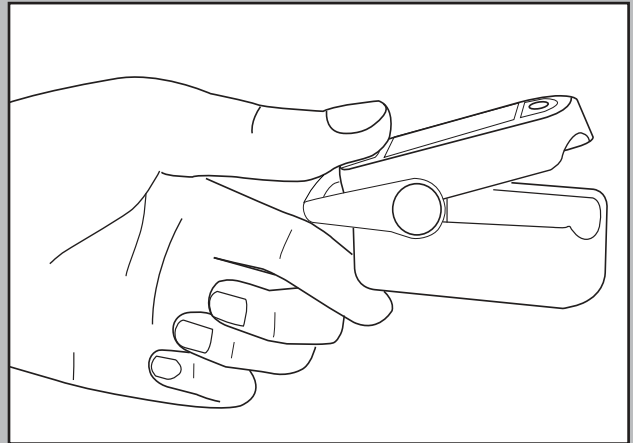


STEP 2. Open the clamp.

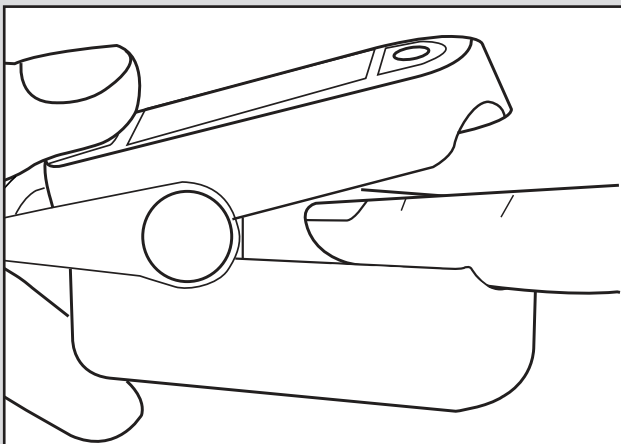


STEP 3. Insert one finger, ideally the index finger, into the opening of the pulse oximeter, please ensure that the patient's nail is facing upwards.

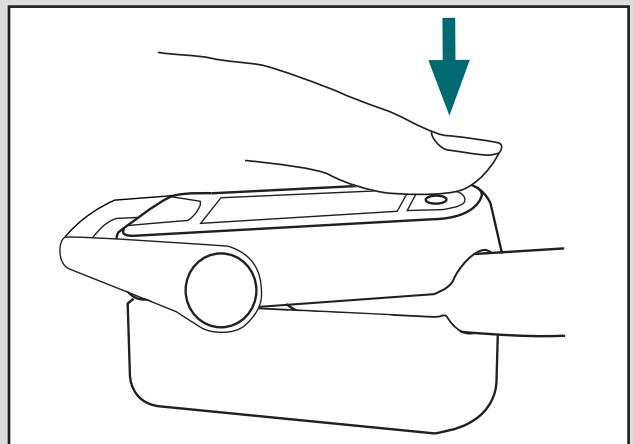

STEP 4. Press the power switch once on the front panel.

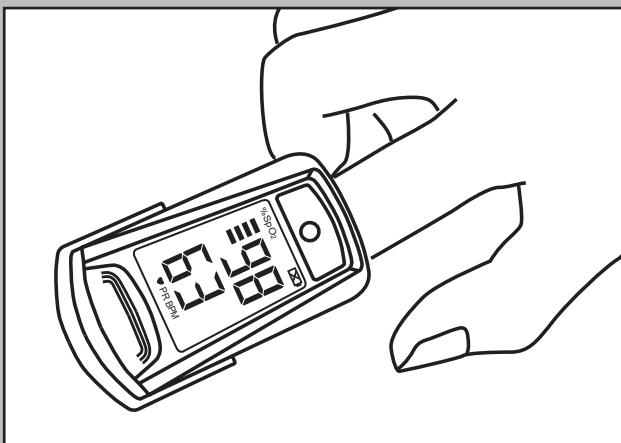

STEP 5. The patient's finger should remain still while the pulse oximeter is working.

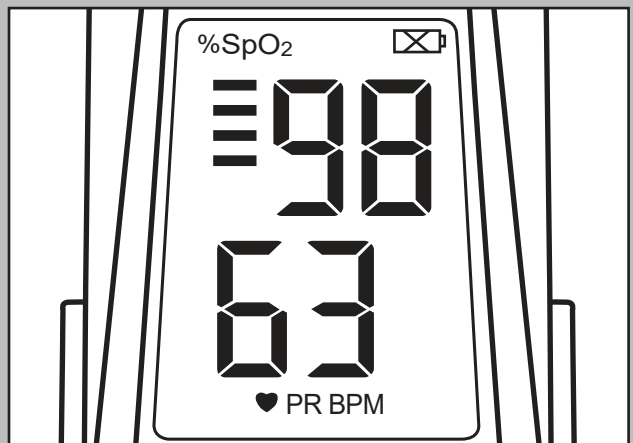

STEP 6. After 2+ seconds, read the measurements from the display screen.

Please visit **riester.de** for more information



Refer to instructions for use for full functional usage



Rudolf Riester GmbH P.O. Box 35 | Bruckstraße 31 | 72417 Jungingen | Germany
Tel.: (+49) 7477-9270-0 | Fax.: (+49) 7477-9270-70 | info@riester.de | www.riester.de

Riester