



NOBAMED Paul Danz AG

NOBAHEBAN®

REF 455407

Product Description, Intended use, Application

The blue cohesive compression bandage (size 7.5 cm x 4.5 m) has a stretchability of approx. 100 %. It is used for bandages for short-term compression. It is applied over a gauze ball or swab. It is also used as a supporting and relieving bandage

Composition

Polyamide, polyester, latexfree elastomere

Contraindications

The product should not be used in the case of a known allergy against the material.

Note

During application, cohesive bandages should not be stretched too much, as otherwise the arterial blood supply might be disturbed.

Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Normative and Regulative Requirements, Common Standards

Medical Device according to *MDD 93/42/EEC, MDR (EU) 2017/745*.

The product does not contain dangerous toxic substances according to REACH.

Packaging

Primary: foil

Secondary: folding box
made of cellulose

Tertiary: carton made of
cellulose

Symbols used in labelling

Explanation at www.nobamed.com



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1- and DIN EN ISO 20417

Storage and Transport

Dry and dustfree

Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.