


# RVS-100

## Vital Signs Monitor


## Quick Start Guide

### APPEARANCE


#### Front View

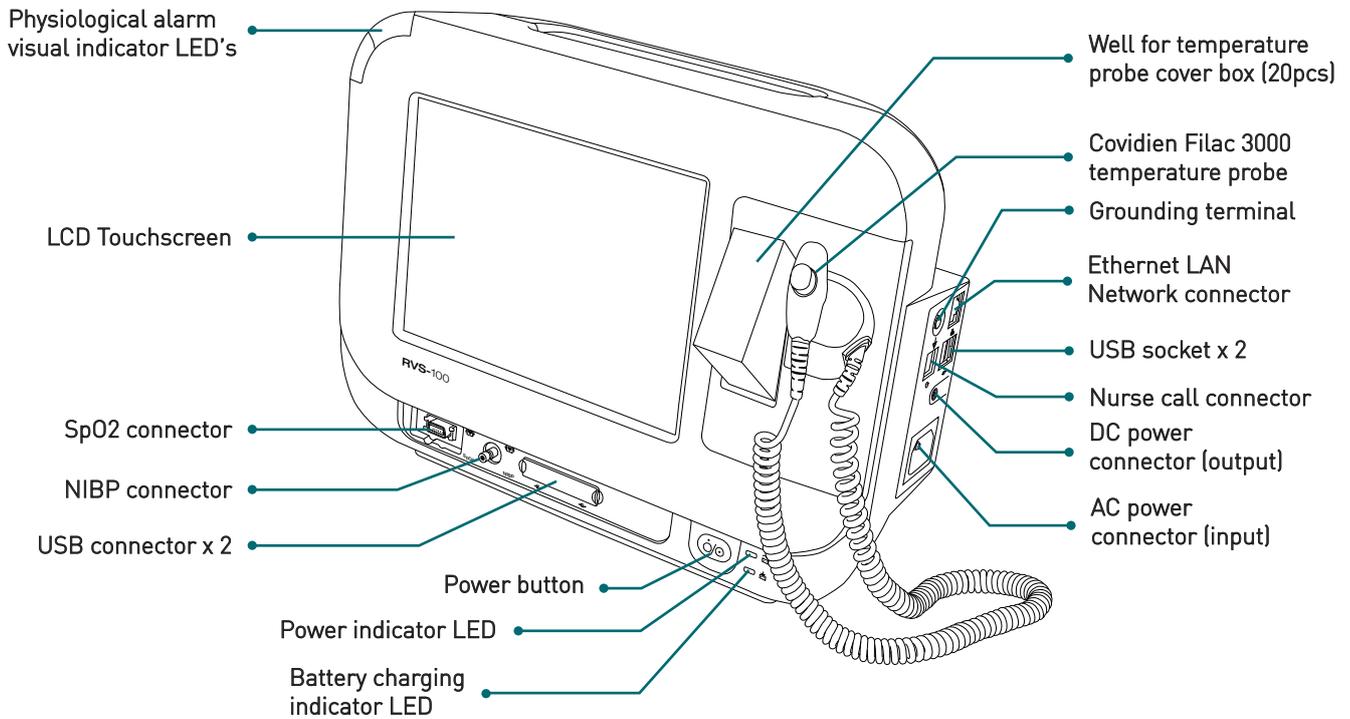


#### Left-side View

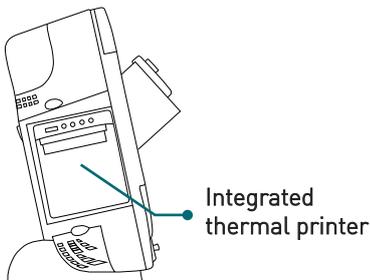

#### Rear View

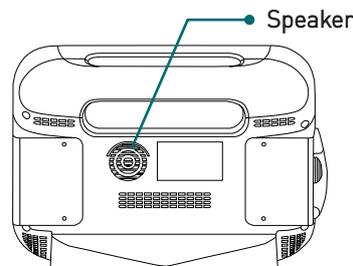

#### Bottom View

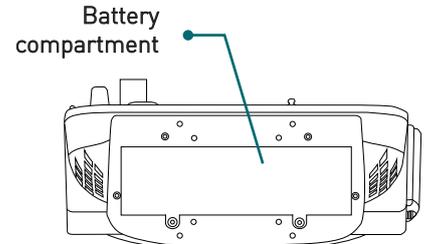

### ORDERING INFORMATION

1960-RRXXE	RVS-100, NIBP (Riester), SpO2 (Riester), EU plug
1960-RRBXE	RVS-100, NIBP (Riester) + SpO2 (Riester) + Pred. Temp, EU plug
1960-RRBPE	RVS-100, NIBP (Riester), SpO2 (Riester), Printer, EU plug
1960-RNXXE	RVS-100, NIBP (Riester), SpO2 (Riester), Pred. Temp, Printer, EU plug
1960-RNBXE	RVS-100, NIBP (Riester), SpO2 (Nellcor), Pred. Temp, EU plug
1960-RNBPE	RVS-100, NIBP (Riester), SpO2 (Nellcor), Pred. Temp, Printer, EU plug
1960-RMXXE	RVS-100, NIBP (Riester), SpO2 (Masimo), EU plug
1960-RMBXE	RVS-100, NIBP (Riester), SpO2 (Masimo), Pred. Temp, EU plug
1960-RMBPE	RVS-100, NIBP (Riester), SpO2 (Masimo), Pred. Temp, Printer, EU plug
1960-RRXPG	RVS-100, NIBP (Riester), SpO2 (Masimo), Printer, UK Plug

# DISPLAY SCREEN

## VITAL SIGNS DEVICES

**Clinician Information:** Department and ID  
**Clinician Information:** Full Name

**Device Alarm Message Bar**

**System Time and Date**  
**Network status**  
**Battery Status**

**Measurement Display Area**

**Patient Information**  
**Menu Tabs**  
**Help key**  
**Alarm pause key**

**Save button**  
**Shortcut key to home screen**  
**Shortcut key to standby mode**  
**Shortcut key to reset the alarm**  
**Shortcut key to print**  
**Shortcut key to start/stop NIBP measurement**

DEMO jim green R000 B000 09:49 2016-08-22  
 1/4 \*\* PR.High  
 NIBP SYS/DIA mmHg 117/79 Manual 92 MAP  
 SYS 120 DIA 93 PR/min 64  
 70 40 30 25  
 Source: SpO2  
 Temp °F 99.3 37.4 °C  
 SpO2 % 99 Normal  
 PATIENT: demo patient Male 22 Pediatric H:180.0cm W:140.0kg  
 MEASURE PATIENT REVIEW ALARM SETTINGS  
 ? [Alarm] [Print] [NIBP] [Standby] [Home]

### NIBP Display

**Pressure unit**  
**Systolic blood pressure/ Diastolic blood pressure**  
**Mean arterial blood pressure**

**Measurement mode**  
**Upper alarm limit**  
**Alarm switch**  
**Lower alarm limit**  
**Patient type**

NIBP SYS/DIA mmHg 117/79 Manual 92 MAP  
 SYS 120 DIA 93  
 70 40  
 Adult

### Pulse Rate (PR) Display

**PR unit**  
**PR value**  
**PR source**

**Upper alarm limit**  
**Lower alarm limit**

PR /min 64  
 30 25  
 Source: SpO2

### SpO<sub>2</sub> Display

**SpO<sub>2</sub> unit**  
**%SpO<sub>2</sub> value**  
**Pleth Waveform**

**Upper alarm limit**  
**Lower alarm limit**

SpO<sub>2</sub> % 99 Normal  
 97 96

### Temperature Display

**Temp unit of measure**  
**Temp value**  
**Alternate Temp value**

**Probe Cover indicator**  
**Measurement site**  
**Upper alarm limit**  
**Lower alarm limit**  
**Mode Key:**  
 Rabbit icon = quick mode  
 Snowflake icon = cold mode  
 Tortoise icon = monitor mode

Temp °F 99.3 37.4 °C  
 90.3 86.0

## USING SPOT MODE



Spot mode Screen




Connecting data on Spot mode Screen

- STEP 1.** After pressing the power switch, the monitor will begin an automatic self-diagnostic test and start-up. During this process, the monitor will check the battery status and for error messages etc. The device will make a sound and the Riester logo will appear on the screen.
- STEP 2.** After the Riester logo disappears, the monitor will display the main user interface. Spot mode is designed for taking a single set of vital signs measurements on a patient. Patient information can be entered and managed, and while technical alarms are still available, physiological alarms are disabled.
- STEP 3.** Connect required cables or sensors to the monitor and connect appropriate cables or sensors to the patient. Ensure that device settings and the installation of cables or sensors are correct.
- STEP 4.** Connecting data will automatically appear on the screen.

## USING NIBP MODE



NIBP mode Screen

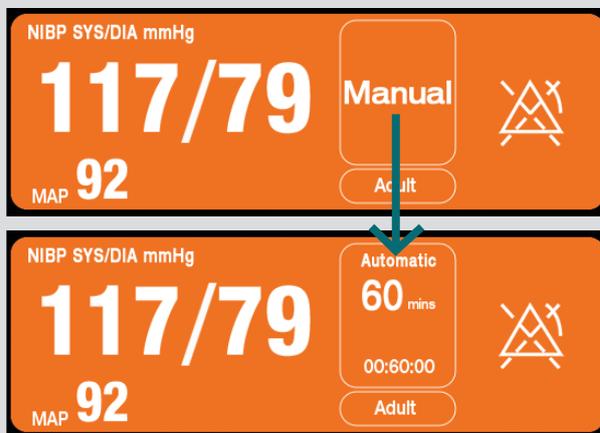

Duration time on NIBP display

- STEP 1.** Press the  button on the device display to read the blood pressure.
- STEP 2.** Press the  button on the NIBP display. There are four modes of measuring NIBP.
  - > **Single Measurement (Manual):** a single measurement on demand.
  - > **Long-Term Automatic:** continuous repeated measurements with a set interval.
  - > **Averaging:** a set number of measurements taken and averaged.
  - > **Short-Term Automatic (STAT):** rapid series of measurements over a five-minute period. For use only on supervised patients.
- STEP 3.** Press the Long-Term Automatic button, and touch the Minute bar to change time. A separate screen will come up on the right side of screen.
- STEP 4.** Type in numbers the duration of time you want to automatically measure blood pressure. Select  button.
- STEP 5.** Select  →  to apply your settings. It will automatically return to the home screen where the new duration time of NIBP will be displayed.

## USING MONITOR MODE




Settings > Profile Screen




Monitor mode Screen

- STEP 1.** On the home screen, press the **SETTINGS** tab on the bottom and **PROFILE** tab on the top to select the work mode you want.
- STEP 2.** Press the Monitor profile button, select **Apply** → **OK** to apply your settings.
- STEP 3.** It will automatically return to the home screen in Monitor mode. Monitor mode is designed for monitoring patients over time, and includes physiological and technical alarms.
- STEP 4.** Press the alarm button **ALARM**, this will open the Alarm Settings for that measurement (i.e. NIBP) where the alarm limits can be adjusted.

## USING ALARM LIMIT

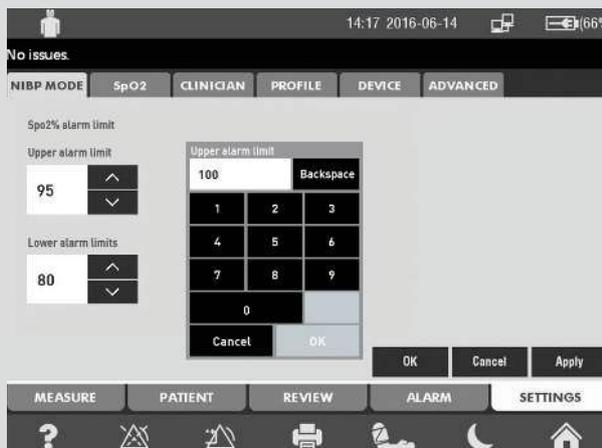

Alarm limit settings

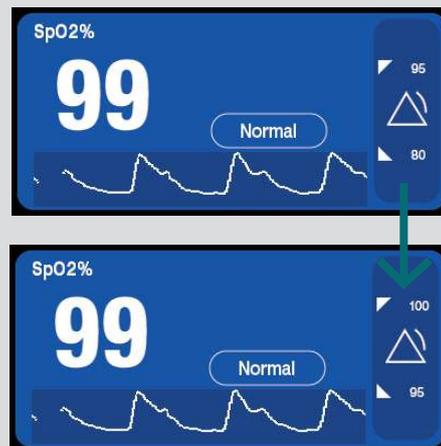

Alarm limits on Spo2% display

- STEP 1.** Go to **SETTINGS** → **PROFILE** and select Monitor to make sure the device is in Monitor profile.
- STEP 2.** From the home screen, press the  icon to access the Alarm Settings.
- STEP 3.** In the Alarm Settings, select the parameter you'd like to change (i.e. NIBP), press the up and down icons to change the limit settings. Alternatively type in numbers for alarm limit you want.
- STEP 4.** Select **Apply** → **OK** to apply your settings. It will automatically return to the home screen where the new alarm settings will be displayed.

## USING ALARM VOLUME

- STEP 1.** Select Alarm → General → Alarm Volume
- STEP 2.** Choose a desired value from Low, Medium, High.
- PAUSING THE ALARM.** Press the  icon on the home screen.

Please visit [riester.de](http://riester.de) for more information



Refer to instructions for use for full functional usage



Rudolf Riester GmbH P.O. Box 35 | Bruckstraße 31 | 72417 Jungingen | Germany  
Tel.: (+49) 7477-9270-0 | Fax.: (+49) 7477-9270-70 | info@riester.de | www.riester.de

