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PPE Regulation (EU) 2016 / 425 CAT III
FFP3NR

EU Declaration of Conformity: http://www.nobamed.com_search672063

Notified body responsible for EU Type examination and ongoing conformity:
BSI Group The Netherlands B.V.
Address: Say Building, John M. Keynesplein 9, 1066 EP Amsterdam
P.O. Box 74103, 1070 BC Amsterdam | The Netherlands

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Stand der Information: Januar 2022

GB Instructions for use

Product Name: FFP3 NR Particulate Respirators

Model: 672063

Intended Use:

Filtering half masks, as RPE, use filters to remove contaminants from the air being breathed in, and to protect against particles except for escape purposes.

Warning:

- Select the respirator type and determine the suitability of the respirator according to the intended use.
- The respirator shall be abandoned when it is broken, becomes difficult to breathe, or polluted by body fluids.
- This respirator is disposable, do not reuse or use for more than one shift.
- Check the respirator before use. Make sure there are no broken earloops, no holes or other damages on the respirator. If any damage detected, please do not use and replace with a new one immediately.
- The effectiveness will be reduced if facial hair or beard passes under the face seal.
- The respirator should only be used in intended environment. Failure to follow the instruction may cause harm to health or reduce the effectiveness.
- The respirator cannot provide intended protection if the mask is not correctly donned or is damaged.
- Never alter or modify this respirator in any way.
- This respirator does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life.
- The respirator has a shelf-life of 3 years. After the expiry date the respirator must no longer be used.
- Do not use in explosive atmospheres.

Storage:

- The unused masks need to be stored in the environment that is well ventilated, dry and away from light. Keep away from fire, pollutants and potential source of pollution.
- Storage temperature: -20 to 50 °C and less than 80% relative humidity.
- Keep the masks in original packaging boxes during transportation to protect against mechanical damage and contamination.

Limitations of Use:

Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this product in an explosive atmosphere. NOBAPROTECT® may be used when a breathing protection according to EN149: 2001+A1: 2009 is required. The mask is not intended for purposes other than described and may not be used in these cases. This product cannot eliminate the risk of contagion or infection.

The respirator complies with Regulation (EU) 2016/425, and conforms with EN 149:2001+ A1:2009.

Application Instruction:

- Take out the respirator.
- Place respirator over face with the nose clamp sitting upward as shown. Position both head straps, making sure your lower chin is covered.
- Hold the nose clamp to the shape of your nose to ensure it fits your face comfortably.
- Adjust the respirator to a proper position and ensure there is no leakage.
- To test fit: cup both hands over the respirator and inhale vigorously, a negative pressure should be felt inside the respirator. If air leaks at the edge, reposition the respirator for better fit.

Manufacturer: NOBAMED Paul Danz AG Höltkenstrasse 1-5 D-58300 Wetter www.nobamed.com	Notified Body: 2797 BSI Group The Netherlands B.V. Address: Say Building, John M. Keynesplein 9, 1066 EP Amsterdam P.O. Box 74103, 1070 BC Amsterdam The Netherlands
--	--

Pictograms:

	Manufacturer		Single use
	Manufacturing date		Humidity Limitation
	Expiry date		Temperature Limit
NR	Not to be used for more than one shift		Keep Dry
	See information supplied by the manufacturer		Keep away from direct sunshine

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